



AcuCap Closure

What is an AcuCap Closure?

The AcuCap Closure is designed to provide accurate metered dose dispensing of pastes, ointments, creams and gels. When used with compounded medications for topical or transdermal delivery, the disposable AcuCap Closure simply takes the place of an ordinary dispensing container cap.

How to Use AcuCap



Align the revolving chamber with the tube outlet and squeeze the tube



Rotate the chamber 180 degrees





Squeeze the tube again

Detailed Instructions

1. Tap the tube a few times to ensure all the cream is at the top of the tube (near the opening).
2. Remove the cap and screw on the AcuCap closure device.
3. Attach the tube winder to the base of the tube and roll up until you feel a small amount of tension.
4. Align the revolving chamber of the AcuCap with the tube outlet and squeeze the tube until the chamber fills.
5. Rotate the chamber 180 degrees (1/2 turn).
6. Squeeze the tube again to dispense the cream.
7. Leave the AcuCap device on the tube until the tube is empty and continue to roll up the tube winder as needed.
8. If the AcuCap device becomes clogged or difficult to use, remove and clean with warm water. Once dry place back on the tube of cream. We recommend you keep the original cap and use when traveling.

Dispensing Key

 Purple cap – dispenses 0.5 gm (1/2 ml) dose with each turn of a rotating chamber.

 Burgandy cap – dispenses 0.25 gm (1/4 ml) dose with each turn of a rotating chamber.